COULTER® RINSOL CLEANING AGENT

For *In Vitro* Diagnostic Use

INTENDED USE

For use as a cleaning agent on the COULTER CellPrep Workstation. For use only with ISOFLOW™ sheath fluid and COULTER CLENZ® cleaning agent. Refer to your COULTER CellPrep Manuals.

SUMMARY

RINSOL serves as a cleaning agent on the COULTER CellPrep Workstation. It is a cyanide-free reagent composed of sodium sulfite and quaternary salts. RINSOL in combination with COULTER CLENZ cleaning agent is used to minimize the carryover of erythrocytes when using whole blood samples.

PRINCIPLE

The CellPrep workstation is an automated system to streamline preparation of cells. It features hollow-fiber technology with the ability to process whole and lysed blood. To maintain filter effectiveness, periodic cleaning of accumulated RBC buildup within the hollow fiber filter is necessary. After each sample, the hollow fiber cartridge is washed with COULTER CLENZ and RINSOL cleaning agents to remove cells, debris and fluorescence reagents.

REACTIVE INGREDIENTS

Quaternary Ammonium Salts. 5-80 g/L
Sodium Sulfite 1-5 g/L
Stabilizer
Buffer

WARNINGS

Do not inhale and/or ingest. Avoid eye and skin contact. In case of eye or skin contact, flush affected area with copious amounts of water for at least 15 minutes.

DO NOT REUSE CONTAINER.

STORAGE, STABILITY AND DISPOSAL

Store COULTER RINSOL cleaning agent at 2 - 40°C. Use at temperatures stated in the instrument product manuals. Do not use product past expiration date. Opened containers are stable for 60 days when stored as recommended. Dispose of waste product, unused product and contaminated packaging in compliance with federal, state and local regulations.

PREPARATION

Mix product by gentle inversion prior to use. If product has been partially or completely frozen, allow product to warm to room temperature before mixing. Prime reagent before use.

REFERENCES

U.S. Patent Document, Number 5,834,315, Number 4,962,038, and Number 4,485,175.
Product Manual, Beckman Coulter Cell Analysis 2003 Featuring COULTER® and Immunotech Products, Bulletin 9275, Page 55.

PRODUCT AVAILABILITY

1 x 5 L COULTER RINSOL Cleaning Agent REF 8547198
US Patent No. 5,834,315; 4,485,175; 4,962,038

TRADEMARKS

Beckman Coulter Logo, COULTER, COULTER CLENZ, and ISOFLOW are trademarks of Beckman Coulter, Inc.

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RÉFÉRENCES

U.S. Patent Document, Number 5,834,315, Number 4,962,038, and Number 4,485,175.

Product Manual, Beckman Coulter Cell Analysis 2003 Featuring COULTER® and Immunotech Products, Bulletin 9275, Page 55.

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1 x 5l Agent de nettoyage COULTER RINSOL REF 8547198
US Patent No. 5,834,315; 4,485,175; 4,962,038

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All Rights Reserved. / Tous droits réservés. / Tutti i diritti riservati. /
Beckman Coulter Ireland Inc.
© 2003 Beckman Coulter, Inc.